
AC00050

SINGLE USE SET DIRECTIONS FOR USE

Language	Instructions (DFU)	Page
English	Directions for Use	2
Italiano	Istruzioni per l'uso	4
Español	Instrucciones para el uso	6
Nederlands	Gebruiksaanwijzing	8
Deutsch	Gebrauchsanweisung	10
Português	Instruções de Utilização	12
Français	Instructions d'utilisation	14
Русский	Руководство по эксплуатации	16
Svenska	Användarinstruktioner	18
Polskie	Instrukcje stosowania	20
Ελληνικά	Οδηγίες χρήσης	22
Română	Instrucțiuni de Utilizare	24
Slovenský	Návod na použitie	26

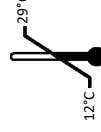This page intentionally left blank.

CardioRenal Systems, Inc.
459 Fortune Boulevard
Milford, MA 01757
508-541-8800

RenalGuard Solutions EU GmbH
Borsteler Chaussee 55
22453 Hamburg, Germany
+49 40 450 3220

AC00050 RenalGuard® Single Use Set ~ Directions for Use

Refer to the RenalGuard Console Operator's Manual for Indications for Use

Warning: *RenalGuard Infusion set can only be used with the RenalGuard Console*

Connecting the RenalGuard® Single Use Set to the RenalGuard® Console

Follow these steps to load the RenalGuard Single Use Set onto the console:

For instructions about priming, changing, emptying and clinical use of the RenalGuard Single Use Set reference the RenalGuard Console Operator's Manual.

1. Visually examine the RenalGuard Single Use Set to determine that the package has not been opened or damaged during shipping. Once opened, preserve sterility.
2. Open the pump door and load the pump segment tubing over the rollers. Line up the loading guide markers **1** and **2** on the infusion set with the loading labels **1** and **2** on the console. Assure that the clips on ends of the raceway are lined up with the tube segment. Close the pump door firmly until you hear a click. Check that the tubing is aligned correctly. Ensure that the tubing from the infusion bag side enters on the left side of the pump.
3. Line up the loading guide marker **3** on the infusion set with the loading guide **3** label below the air detector on the front of the console. Insert the tubing into air detector channel, ensuring that the tubing is firmly seated.
4. Plug the transducer connector marked **4** on the infusion set into the pressure transducer receptacle (labeled **4**) on the front of the console. Ensure that the connector is fully seated.

Caution: *The RenalGuard System™ is designed for use with 1 (one) liter infusion bags only. The system will not operate properly with other sized infusion bags.*

5. Using the standard hospital technique, spike a **1 liter infusion bag** with the infusion set bag spike.
6. Facing the front of the console, hang the full infusion bag on the LEFT chain hook. Ensure that the bag hangs freely and that its motion will remain unobstructed as it empties.
7. Facing the front of the console, hang the empty collection set urine bag on the RIGHT chain hook. Ensure that the chain supporting the urine bag passes through the ring of the chain guide assembly to minimize excessive swinging of the bag. Ensure that the bag hangs freely without touching the floor and that its motion will remain unobstructed as it fills.
8. Insert the collection set tubing into the notch on the front of the chain guide assembly. Align the Yellow loading guides on the tubing to the right and left of the notch to prevent the tubing from pulling on the urine collection bag.
9. Confirm that the drain valve at the bottom of the urine collection bag is fully closed.

Warnings: *The RenalGuard System™ will not operate properly if the urine bag is not hanging freely on the weight scale chain, or is interfered with in any way.*

Prime the circuit in accordance with the RenalGuard Console Operator's Manual instructions.

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Connecting the RenalGuard® Single Use Set to the patient

Warning: Aseptic technique must be used throughout the procedure. Usual and customary hospital infection control practices should be followed when replacing the patient connected tubing to ensure the safety of the patient and clinician.

1. Ensure that both the Urinary Catheter and the I.V. Catheter have been successfully inserted into the patient and are prepared for use. The Urinary Catheter and I.V. Catheter should be flushed and flowing properly prior to connecting the single use set.
2. Disconnect the infusion set Luer connector from the RenalGuard prime adapter on the collection set. Remove the prime adapter which is used only during the priming procedure. The adapter may be discarded at this time.
3. Connect the infusion set Luer connector to the inserted I.V. Catheter. Ensure that no air enters the system. Turn the Luer connector until it is completely closed. **DO NOT OVERTIGHTEN.**

Warning: Automatic air detection and occlusion detection are disabled during manual connection operations.

4. Visually check for air in the lines.

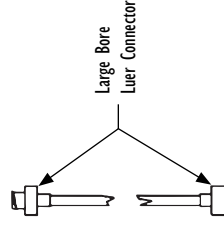
If air is noted, disconnect the infusion set from the inserted I.V. Catheter and use the ADVANCE mode to purge the air:

- Press and hold the Advance button. Once the Advance button is released, the pump will stop in 2 seconds.
- The Advance button cannot be depressed for longer than 30 seconds or the pump will automatically stop.
- Releasing the button and then pressing and holding it again will allow the pump to run for an additional 30 seconds.
- 5. Connect the collection set's male Foley connector to the inserted Urinary catheter. Ensure that the connection does not leak.

For instructions about priming, changing, emptying and clinical use of the RenalGuard® Single Use Set, reference the RenalGuard® Console Operator's Manual.


Each RenalGuard® Single Use Set includes:
Infusion Set
Collection Set
Extension Set

Extension Set

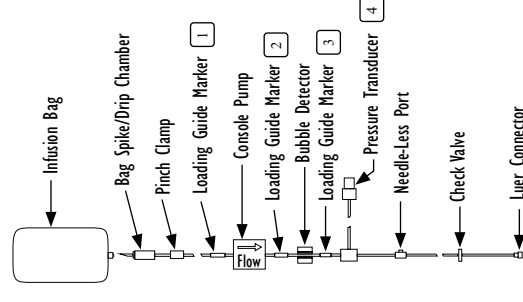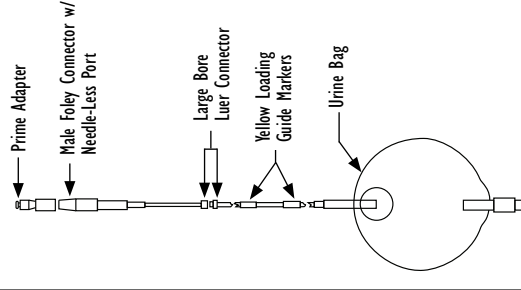
Warning: After use, dispose of sets in accordance with local regulations and/or protocols

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